



CATALOGUE



PhysiPod

BY PROFESSIONALS FOR PROFESSIONALS

WELCOME

About Us

PhysiPod is a business trading out of Melbourne, Australia that has been established with a commitment to supplying the Allied Health Industry with a 'niche' range of high quality, reasonably priced consumables for physiotherapists, podiatrists and related Allied Health practitioners.

PhysiPod's owners have worked in the physiotherapy and podiatry professions for the last 24 years.

Years of practising as clinicians, has allowed us to leverage considerable knowledge from both professions.

This, in turn, has led to the research and development of several new and innovative products for the physiotherapy, podiatry and allied health sector, that have been designed – By Professionals For Professionals.


We hope you find our products superior in design, performance, durability, quality and cost.

Our Orthotic range is a unique, and innovative design that is aimed at reducing the Clinician's hands-on time and the client's cost. Similarly, our Latex range is unique, time saving and versatile enough to be used for almost all rehabilitation applications. As well as general exercise and muscle strengthening.

In short, the PhysiPod range are essential items we would recommend for all physiotherapists, podiatrists and related Allied Health practitioners.



EXERCISE EQUIPMENT

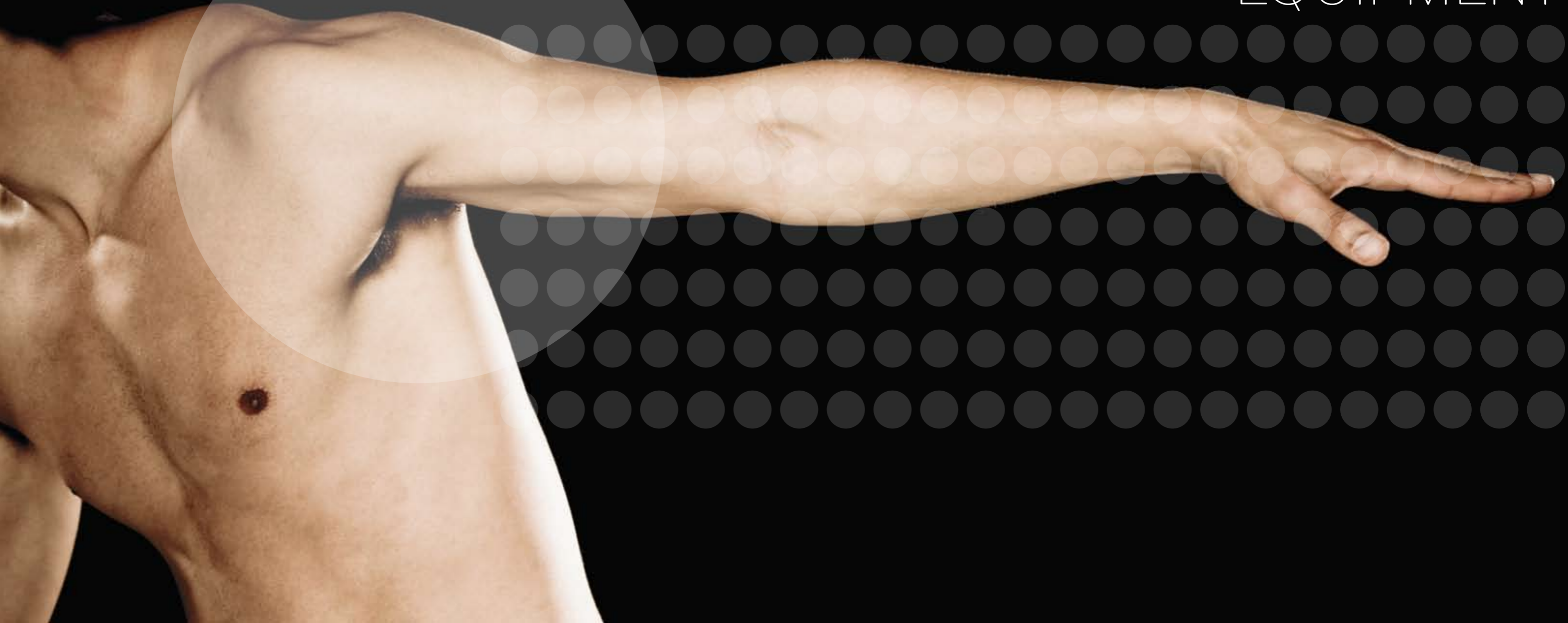

EXERCISE EQUIPMENT

Latex Physiotherapy products

The PhysiPod Latex Range is specifically designed for clinicians to have versatile, functional and interchangeable items ready at your fingertips for all your rehabilitation needs.

These kits are professionally presented with easy to follow exercises included inside the pack. Patients can retain these exercises for future reference. They can also be easily modified for specific, speedy, exercise prescription.

The PhysiPod Latex Exercise Range is conveniently colour coded according to resistance strength across the whole of the latex family of products. Choose from single banding or tubing in four grades of resistance – Light, Medium, Heavy, X Heavy. Kits are designed for both upper and lower limb use, or bilateral limb use, via the Pilates Kit.

New innovations in Latex

The PhysiPod Latex range is made with the highest quality, powder-free latex. Our range has premium quality additions such as the interchangeable door/table attachment. This very durable attachment allows use at any height from floor to the top of door. It can also be attached to a table leg, upright pole, or a door jam.

The latex then simply clips on and off the D-rings of the door attachment. For all lower limb applications an ankle cuff attaches to the door attachment.

The soft neoprene cuff wraps comfortably around the ankle, and can also be used on the upper arm for those subtle, small amplitude shoulder exercises. All clips, webbing and attachments are high quality, robust and customised for comfort, durability and versatility.

Functional & Convenient

For all physiotherapists using banding or tubing, these are a functional and convenient addition to your treatment regime. Prescription of a Home Exercise Program just became so much easier.

To progress from a Pilates Reformer to a Home Exercise Program is often difficult to replicate. The PhysiPod Pilates Kit solves this problem making it easy to replicate exercises in kneeling, standing, four foot kneel, supine, single kneel stance and so on. And with the addition of a lightweight stick to the Pilates Kit, a new dimension of exercise becomes possible, just like that.

Easy to follow diagrams

Each latex product has its own exercise chart with pictorial and written instructions of sample exercises. This saves the clinician time and effort and is a more professional approach to exercise prescription.

Imagine, no more time consuming (and challenging) stick figure drawings!



PRODUCT CODE AH-L05-M
PRODUCT CODE AH-L05-H
PRODUCT CODE AH-L05-XH

PhysiPod Pilates Home Kit


The PhysiPod Pilates Kit has been designed to allow you to progress your 'in rooms' Pilates regime to the home environment seamlessly. No need to buy expensive equipment when you can replicate these exercises with this Pilates Kit. Used alone, or with the addition of a lightweight stick, this kit can assist with many upper and lower limb exercises and with a variety of core strengthening and stability exercises as well.

The dual tubing comes with incorporated end loops, which can be used either by hand or foot. The tubing clips easily on to the overdoor/table attachment for use at floor, waist, or above shoulder height. This multi-tasking attachment can be fastened to a table, doorway, or even a pole, allowing a pilates routine to be performed almost anywhere.

CONTENTS:

1 x Door/Table Attachment
1800mm Dual Tubing
with Clips & Footstraps



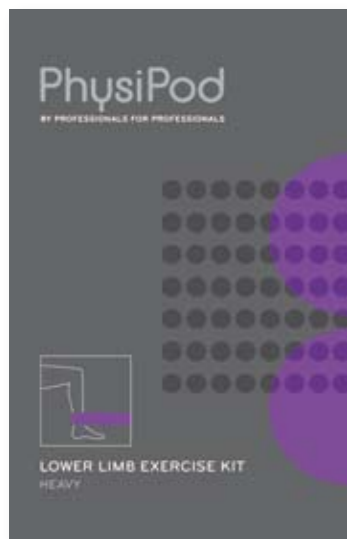

PRODUCT CODE AH-L04-M
 PRODUCT CODE AH-L04-H
 PRODUCT CODE AH-L04-XH

PhysiPod Lower Limb Exercise Kit


The PhysiPod Lower Limb Exercise Kit is a convenient, versatile targeted assembly, designed primarily to help strengthen the lower limb. It can also be used for the upper limb as an upper arm cuff, or held in the hand to exercise the upper limb.

The double layer of soft neoprene in the cuff softly cradles the ankle, without any unpleasant pressure around painful, swollen or Diabetic ankles.

This kit also has the extra bonus of the overdoor/table attachment. This can be used above shoulder height, at waist height, at floor level, or around a table leg or upright pole. Thus making this kit easily transportable to the workplace, at home, or on holidays. Exercising has never been so portable or as much fun!

CONTENTS:

- 1 x Soft Neoprene Premium Ankle Strap/Cuff
- 1 x Door/Table Attachment
- 1000mm Exercise Tubing (detachable/interchangeable)

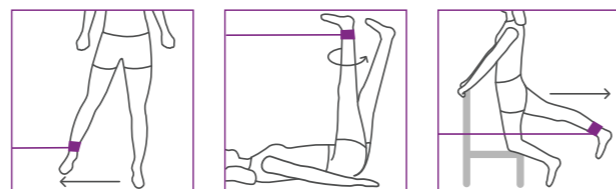

PRODUCT CODE AH-L02-L
 PRODUCT CODE AH-L02-M
 PRODUCT CODE AH-L02-H
 PRODUCT CODE AH-L02-XH

PhysiPod Overdoor Exercise Kit

This kit is designed for the ultimate in versatility and efficiency with latex exercises. The kit contains latex tubing, a handle and door attachment. Clients simply choose the level of resistance they are ready for and off they go. The premium grade handle allows clients to add to the kit if required by using a more difficult grade of latex once the resistance level is achieved. The handle simply and quickly detaches, allowing the addition or subtraction of resistance levels.


The overdoor attachment allows clients to exercise with freedom of position and freedom of location.

It can be attached above shoulder height, at waist level, floor level, through a doorway, or around a table leg or pole. It can thus be used almost anywhere an anchor point can be gained. No excuses for not taking your exercises with you on that holiday, or to work.

Improve your strength, stability, range of motion and co-ordination. Sample exercises are included for your convenience.

CONTENTS:

- 1 x Door/Table Attachment (interchangeable)
- 1 x Premium Exercise Handle
- 1200mm Exercise Tubing

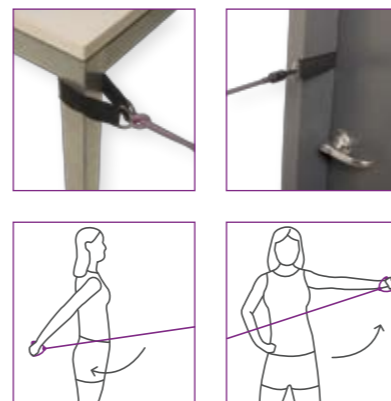



PRODUCT CODE AH-L01

PhysiPod Latex Band Exercise Kit

The PhysiPod Latex Exercise Band is designed to assist in the rehabilitation of injured or weak muscles. A simple yet versatile piece of equipment available in four different strengths to progressively and safely improve muscular strength and endurance. Helpful in training core stability, range of motion and improving posture and co-ordination.

Sample exercise instructions and diagrams are included.

CONTENTS:

- 1 x Handle
- 3 x Exercise Banding



PRODUCT CODE AH-L03

PhysiPod Premium Handle

The PhysiPod Premium Exercise Handle is designed to co-ordinate and interchange with all of the PhysiPod Latex range. The latex band, or tube, is quickly and easily removed from the handle, allowing the interchange of different resistances for different applications.

It is the strongest handle on the market, yet sleek, light and versatile. It has a thick EVA padding around the handle, making it ideally suited for the smaller hand, the elderly hand, the post-op hand, or for the average hand. The hand grip pivots, allowing for quick and easy position changes whilst exercising.

Your client can take it anywhere and it will keep their goals on track.





PRODUCT CODE AH-B01-4

PhysiPod Latex Exercise Band

More convenient than rifling through the cupboard and cutting a jagged piece of band from a 10m roll. These pre-cut 1200mm lengths are purpose designed for all your banding needs.

They are beautifully presented in a neat sleeve pack, in four shades of purple (lightest to darkest = easiest to heaviest resistance).

Help make your clients rehabilitation easier and more comfortable. Sample exercises enclosed.

CONTENTS:
1200mm Latex Band



PRODUCT CODE AH-T01-4

PhysiPod Latex Exercise Tube

Again, more convenient than cutting from a bulk pack, these individually packaged 1200mm pieces of tubing also come in four colours (lightest to darkest = easiest to heaviest resistance).

The tubing can be purchased separately, or in conjunction with either the Premium Handle or any of the other kits. These are designed for quick and efficient exercise prescription, sample exercises enclosed.

CONTENTS:
1200mm Latex Tubing



PhysiPod Balance Board

The sleek and modern design of the PhysiPod Balance Board will make your clients want to exercise on it all day long. This hard working device will last forever.

Extremely strong and stable in design, with a superior non slip surface, it will be perfect for all balance retraining and core

strengthening needs. The surface has been specially designed with grooves and texture, to increase traction and grip. Excellent for rehabilitation, strength, core stability and range of motion. Available in green and purple.

We think your clients will have fun on this whilst they achieve their rehabilitation goals.

PRODUCT CODE AH-BB-G-P



SOCKS



SOCKS

PhysiPod socks have been designed specifically for the swollen, painful, or Diabetic foot, however, they are equally suited for everyday use on the average foot. They have been manufactured using the highest quality natural cotton, to offer comfort, durability, breathability, and absorbency. The socks have a small percentage of spandex to help the sock retain its shape over time.

When distal blood flow is compromised, or oedema is present, the PhysiPod sock is designed to not restrict blood flow any further. The sock's design has graduated shaping along the shin, progressing to a loose knitted top, with a soft cuff. Making them perfect for problem feet!

The sock's footbed has a full terry base which cushions the foot, further aiding comfort, absorbency and ventilation. A fine hand finished seam at the toe and heel, avoids irritation of sensitive skin.

Practical and purpose designed for maximum comfort, foot pain or no foot pain, your clients will come back for PhysiPod socks!



PRODUCT CODE AH-DS-B
 PRODUCT CODE AH-DS-N
 PRODUCT CODE AH-DS-BR
 PRODUCT CODE AH-DS-W





PhysiPod Circulation Socks


PhysiPod socks are recommended for the Diabetic and feet prone to swelling or circulation problems. Designed for ultimate comfort, PhysiPod socks can be worn for all foot types.

Available in four colours, White, Black, Navy and Brown. Sizes are unisex: Small, Medium and Large. Machine washable and fade resistant. PhysiPod socks are perfect for all feet!



COLOURS AVAILABLE

-  BLACK (B)
-  NAVY (N)
-  BROWN (BR)
-  WHITE (W)

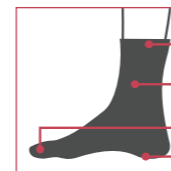

- UNRESTRICTIVE LOOSE TOP
- NATURAL COTTON FIBRES AID IN VENTILATION AND ABSORPTION
- FINE TOE SEAM
- COMFORT PADDED KNITTED SOLE



ORTHOTICS

The PhysiPod Orthotic range has been biomechanically designed to provide functional correction for a variety of foot and lower limb pathologies. Designed by a Podiatrist, using the latest manufacturing technologies and highest quality materials, the PhysiPod range will meet your clinical requirements.

Design features include:


- rearfoot and forefoot posts from 0, 2, and 4 degrees
- molded arch filler to increase medial support
- anti bacterial top cover
- heat moldable
- metatarsal forefoot pad and bar
- heel raise

All standard across the range.

But the true star of the PhysiPod Orthotic range is our innovative Component Orthotic. Designed to save you time, money and put a stop to fiddly, frustrating orthotic customisation. PhysiPod's Component Orthotic revolutionises custom fitting, by allowing you to tailor an Orthotic on the spot using the various components included in the kit, quickly and easily.

Using the self adhesive components, clinicians can add and modify the Component Orthotic from the top and bottom to control and redistribute pressure areas to the foot. The self adhesive top cover then allows for an easy finish to complete the Orthotic. No need for making pads, grinding or messy glue. Your clinical imagination and a pair of scissors are all you need!

PhysiPod Component Orthotic Design Features

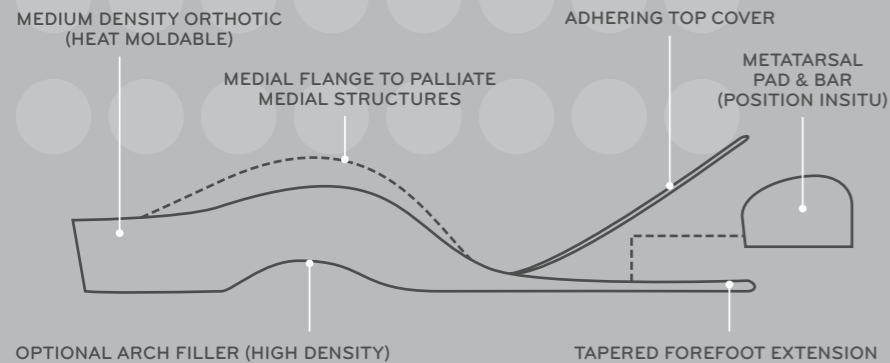


PRODUCT CODE AH-CCO

PhysiPod Orthotics offer additional unique arch filler for added arch support and stability, allowing immediate conversion from medium to firm correction.

PhysiPod Component Orthotic

The Component PhysiPod Orthotic has been developed after many years of Podiatric practitioner experience, to offer a fully customisable Orthotic to suit multiple foot and lower limb complaints. All Health professionals can design the Orthotic to suit their patient's needs with the components supplied. The kit provided offers multiple combinations of mechanical and palliative additions for tailoring the Orthotic.

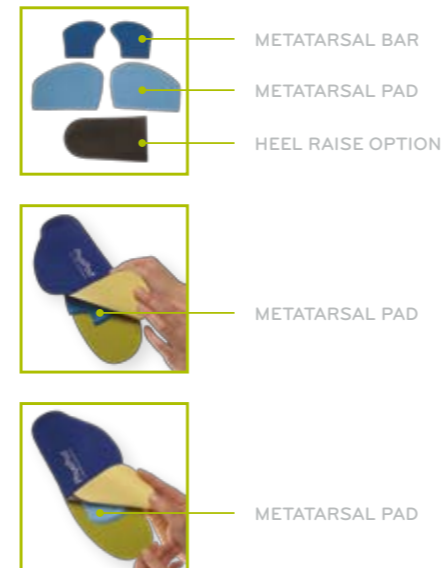

PhysiPod Orthotics have been designed by an experienced Podiatrist to biomechanically correct foot alignment and offer maximum support to the foot throughout the gait cycle. Using the latest engineering, technology and highest quality materials, PhysiPod Orthotics will offer optimum control and comfort in prefabricated Orthotic therapy.

Many foot and lower limb complaints can be treated with PhysiPod Orthotics, including:

- flat feet and over pronation
- plantar fasciitis/heel spur
- metatarsalgia/ball of foot pain
- Morton's neuroma
- inner knee pains
- shin splints
- lower back pains





PRODUCT CODE AH-FLO

PhysiPod Full Length Orthotic

Designed by an experienced Podiatrist, PhysiPod Orthotics biomechanically correct foot alignment and offer maximum support to the foot throughout the gait cycle.


Using the latest engineering technology and highest quality materials, PhysiPod Orthotics will offer optimum control and comfort in prefabricated Orthotic therapy.

Many foot and lower limb complaints can be treated with PhysiPod Orthotics, including;

- flat feet and over pronation
- plantar fasciitis
- heel spur

- metatarsalgia
- ball of foot pain
- Morton's neuroma
- inner knee pains
- shin splints
- lower back pains

PhysiPod Orthotics offer additional unique arch filler for added arch support and stability, allowing immediate conversion from medium to firm correction.

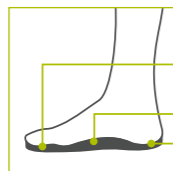

3 REARFOOT & FOREFOOT POSTING OPTIONS

ANTIBACTERIAL TOP COVER
SUPERIOR EVA CONSTRUCTION



PRODUCT CODE AH-TQO


PhysiPod 3/4 Length Orthotic

Just like the full length Orthotic, our 3/4 Length Orthotic has been designed by an experienced Podiatrist to aid the foot throughout the gait cycle, offering maximum support and biomechanically correct foot alignment. Using the latest engineering, technology and highest quality materials, PhysiPod Orthotics will offer optimum control and comfort in prefabricated Orthotic therapy.

PhysiPod Orthotics can be used in the treatment of many foot and lower limb complaints, such as:

- flat feet and over pronation
- plantar fasciitis/heel spur
- metatarsalgia/ball of foot pain
- Morton's neuroma
- shin splints
- inner knee pains
- lower back pains

PhysiPod Orthotics offer additional unique arch filler for added arch support and stability, allowing immediate conversion from medium to firm correction.

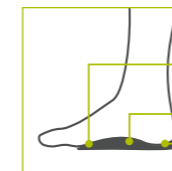

3 REARFOOT & FOREFOOT POSTING OPTIONS

ANTIBACTERIAL TOP COVER
SUPERIOR EVA CONSTRUCTION



TENS

TENS

T.E.N.S., or Trans Cutaneous Electrical Stimulation, has long been used as an effective method of pain relief for chronic, intractable pain and for acute post surgical and post traumatic pain. More recently, it has also been used for obstetric pain relief.

The physiological and therapeutic effects of electrical stimulation when applied to innervated muscle can be summarized as follows:

- Muscle strengthening in healthy and atrophied muscle
- Facilitation of motor control
- Maintenance or improvement of joint range of motion
- Increased muscle metabolism, blood flow, venous and lymph flow
- Reduction of oedema
- Accelerated tissue healing
- Gate effect on both A delta (fast), and C (slow), pain fibres in the posterior horn, due to stimulation of mechanoreceptors .
- Morphine-type effect on the C fibre system.

TENS can be used as an adjunct in the treatment of neurogenic pain; fibromyalgia; post-operative and obstetric pain; for use after sports injuries and in chronic musculoskeletal disorders.

As part of the PhysiPod range of products, we are introducing two new TENS machines to the Australian market – PhysiPod 2 and PhysiPod 4. PhysiPod 2 is a 2 channel combined TENS/NMES machine. The PhysiPod 4 is a 4 channel combined TENS function only model.

Both are of sleek and modern design, with large LCD screens, ensuring accurate output and are suitable for a wide range of applications. Small and compact in design the PhysiPod TENS allow you to take them anywhere, for treatment at any time.

A cost effective solution for patients seeking self-treatment, PhysiPod TENS is a must for your rooms and retail sales.



PRODUCT CODE AH-TU2

PhysiPod 2 Channel TENS

This 2 channel digital combined TENS/NMES unit has 10 pre-set programs (P1-P10) and 10 user adjustable programs (U1-U10), including 6 modes, (on/off; standby; preset mode treatment time user preset mode; program mode; and user adjustable mode). The 2 channels work independently of each other and can work simultaneously.

The default setting is a combined TENS/NMES setting for the relief of pain. It has a belt clip and can be easily attached to one's clothing, so you can move around during treatment.

The large LCD display ensures accuracy and is easy to use.

FREQUENCY: 1-175 Hz

INTENSITY: 0mA-70mA


PULSE AMPLITUDE: 30-300µs

POWER SUPPLY: 4 x AAA Batteries

CONTENTS: TENS unit.

4x AAA Batteries. 2 sets of leads.

2 pairs of electrodes. Instruction manual. Protective carry case.

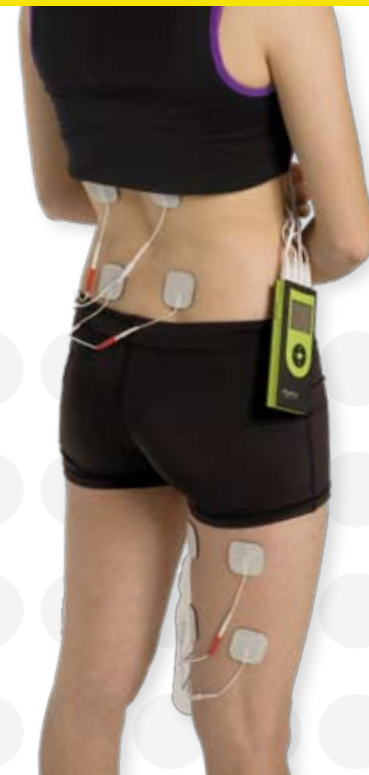

PRODUCT CODE AH-TU4

PhysiPod 4 Channel TENS

This unit is a 4 Channel digital TENS with 42 preset programs, designed to cater for all electrical neuromuscular stimulation applications. The 4 channels can work independently of each other and allow you to treat a larger area in one application. Up to 8 electrodes can be used at one time, thus saving you time and effort when covering larger areas, or multi-site injuries. This unit is easy to use. The clear LCD display, with back lighting, ensures accuracy and ease of program selection.

Each program produces different effects suited to a specific application. It can be used for relaxation; pain relief; to decrease muscle spasm; to improve muscular hypertonicity and strength; and increasing local blood circulation.

FREQUENCY: 2-120 Hz

INTENSITY: 0mA-60mA into 1000ohm load (60 step, each channel adjusted separately)

PULSE AMPLITUDE: 50-400µs

POWER SUPPLY: 4 x AAA Batteries

CONTENTS: TENS unit. 4x AAA Batteries. 4 sets of leads.

4 pairs of electrodes. Instruction manual. Protective carry case.



SILICONE



SILICONE

The PhysiPod silicone range incorporates the most popular silicone foot and digital products. Our range is made from the highest quality medical grade silicone and mineral oil gel, to offer durability, comfort, support and lasting wear. Specific durometers have been chosen for all of our range, to optimise support where required and maximize comfort where needed. They can be used to alleviate pressure from corns, callous, toe deformities and poor fitting footwear.

Silicone Gel

Silicone compounds remain the material of choice for cushioning and support for many health care professionals because of proven performance and reliability.


Our silicone gel products aid in absorbing direct pressure, friction and shearing forces. These products are washable, reusable and hypoallergenic.

Mineral Oil Gel

Equivalent in medical purity to silicone gels, our mineral oil gels offer an enriched formula which exudes a mineral oil throughout the product life. Mineral oil gels hydrate, moisturize and soften the skin, thus making them ideal for a variety of digital and foot applications. The mineral oil gel products aid in absorbing direct pressure, friction and shearing forces. These products are washable, reusable and hypoallergenic.



PRODUCT CODE AH-SHC-M
 PRODUCT CODE AH-SHC-L
 PRODUCT CODE AH-SHC-XL

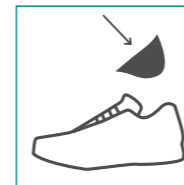

PhysiPod Heel Cups

PhysiPod Heel Cups achieve maximum shock absorption, by offering a true dual density heel cup. The outer cup construction absorbs impact and retains its shape, whilst the soft blue zone maximizes shock absorption and comfort due to its lower durometer. Ideal for all activities such as walking and running.

Medical grade Silicone.

INDICATIONS:

- Plantar Fasciitis
- Heel Pad Atrophy
- Heel/Calcaneal Spur





PRODUCT CODE AH-SPHW-M
PRODUCT CODE AH-SPHW-L

PhysiPod Silicone Posted Heel Wedge

The PhysiPod Silicone Posted Heel Wedge is a dual density wedge for relief of heel spur/fascial discomfort. Unique design incorporates a soft blue gel zone, positioned on medial side to maximize cushioning under the focal tender area. Additional medial posting aids in the prevention of pronation in flat foot conditions. Made of medical grade silicone with dual Durometer silicone zones, to offer maximal cushion and control. Durable and long lasting.

INDICATIONS:

- Heel Spur-Plantar Fasciitis
- Fat pad Atrophy of heel
- Calcaneal apophysitis in children
- Flat Foot conditions



PRODUCT CODE AH-BGS-5

PhysiPod Bunionette Gel Shield


The PhysiPod Bunionette Gel Shield is tailored to protect and palliate the last toe area from friction and pressure from footwear. The thin and contoured shell conforms to the Bunionette and fits in most footwear. Easy application and durable, the PhysiPod Bunionette Shield can be worn on either foot.



PRODUCT CODE AH-SFLI

PhysiPod Silicone Full Length Insoles

PhysiPod Silicone Full Length Insoles offer forefoot and rearfoot comfort and pain relief due to the dual soft zones. A large central heel and anatomically contoured metatarsal region soft zone minimises the impact and shock with walking and running. Vented holes in the insole aid in air flow. Made of medical grade silicone, they are washable and durable.

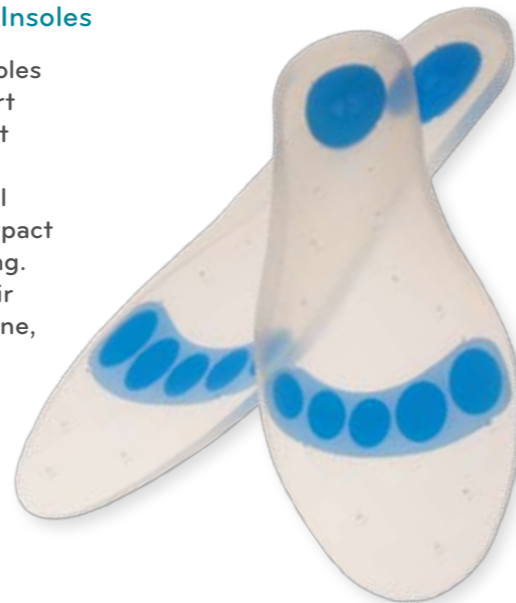

PRODUCT CODE AH-BGS-1

PhysiPod Bunion Gel Shield

The PhysiPod Bunion Gel Shield is tailored for the painful bunion, by offering a shaped and contoured shell to protect and palliate the joint from the shoe. Conveniently thin and tapering, the bunion shield can be worn in most fashion footwear. Easy application and durable, the PhysiPod Bunion Gel Shield relieves friction and pressure from footwear pressure, and may be worn on either foot.






PRODUCT CODE AH-GTW

PhysiPod Gel Toe Wedges

The PhysiPod Gel Toe Wedges are designed to fit between the first and second toe, to straighten and re-align overriding toes and relieve pressure from bunions.

The soft gel palliates the toes and, also interdigital lesions, due to the molded wedge shape. Painful lesions are also softened by the mineral oil properties released to the skin.

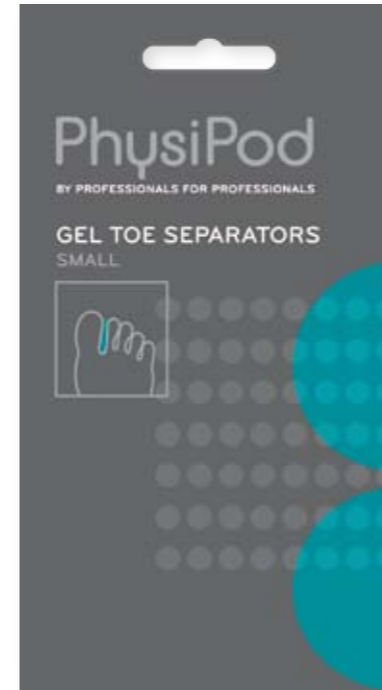

PRODUCT CODE AH-GTS-S
PRODUCT CODE AH-GTS-L

PhysiPod Gel Toe Separators

PhysiPod Gel Toe Separators are designed to separate small toes and relieve pressure and friction from clawed and hammered toes, or tight footwear. They gently separate the toes and absorb pressure, reducing irritation to the toes and interdigital lesions. They are conveniently made and supplied in two sizes.




PRODUCT CODE AH-SDTP-L
PRODUCT CODE AH-SDTP-R

PhysiPod Digital Toe Prop

PhysiPod Toe Props are designed to lift and palliate the apices of the lesser toes. The tapered design follows the web space of the toes and is ideal for clawed and hammered toe deformities.

Made of a medium durometer Silicone, the toe prop eases pain and discomfort to corns and callous by lifting and supporting the toes. Sold separately as left and right.



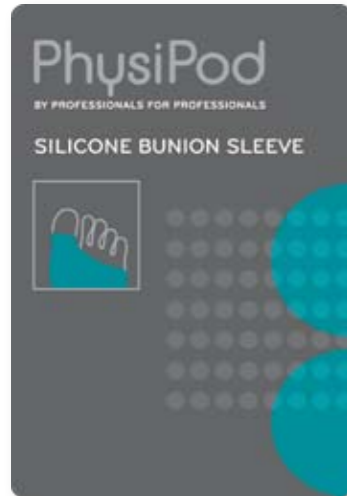

PRODUCT CODE AH-SBS

PhysiPod Silicone Bunion Sleeve

The PhysiPod Silicone Bunion Sleeve offers a soft gel disc within a flesh coloured sleeve to cushion and protect painful bunions. The gel disc is held in place by the soft and conforming elasticised sleeve. Ideal for most footwear due to a new thin design.



PRODUCT CODE AH-GDDP

PhysiPod Gel Digital Dot Pads

PhysiPod Gel Digital Dot Pads are easy to wear fabric covered gel pads, which can be applied to clawed and hammered toes. The elastic fabric cover conforms and stretches to most toes and easily holds the soft gel pad in place. Gel Digital Dot Pads absorb pressure and friction, to cushion and protect from painful corns, callouses and blisters.



PRODUCT CODE AH-SFCS

PhysiPod Silicone Forefoot Cushion Sleeve

The Silicone Forefoot Cushion Sleeve offers a soft gel disc across the whole forefoot, to protect against friction and pressure. The gel disc serves as a soft tissue supplement where there is atrophy of the plantar fat pad or bony prominences. The gel disc is held in place by a soft elasticised flesh coloured sleeve covering the forefoot. Ideal for most footwear due to a new thin design.



PRODUCT CODE AH-DGC

PhysiPod Digital Gel Cap

PhysiPod Digital Gel Caps are designed to protect the apices of fingers and toes from direct pressure. The elastic fabric cover conforms and stretches to most fingers and toes and envelops the digit with a thick soft gel button at the apex. Ideal for use in clawed and hammered toes and fingers, by cushioning the tip of the digit.




PhysiFeet

Today's modern footwear does not always allow for the foot or toes to fit with ease and room to spare. This can lead to blistering of the feet and pressure areas under the forefoot and heel. If pressure areas and blistering occur, PhysiFeet products provide a solution.

We have five products within the PhysiFeet range, all designed to improve comfort in fashion footwear. Not only will PhysiFeet look fabulous in shoes, feet will feel like they can party all night long.

Easy to apply to almost any shoe, the soft gel cushions and dots help prevent and relieve tired and painful feet. The invisible gel is discreet and thin and will be a valuable addition to all fashion footwear. There is even a product to accommodate the discomfort of a thong between the first and second toe. This thong protector alleviates the rubbing commonly associated with wearing thong-like sandals.

PhysiFeet products offer a non-slip surface with an adherent base, ease of fit, durability and outstanding conformity to any shoe. They are washable, reusable and hypoallergenic. Step out with confidence and pain-free feet. With PhysiFeet, it's time to party!

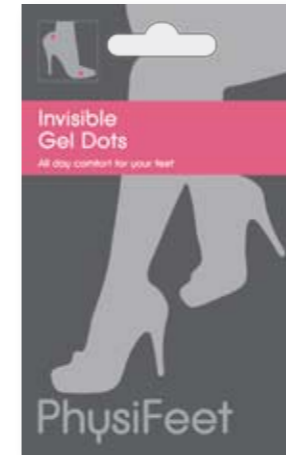

PRODUCT CODE AH-IGD

PhysiFeet Invisible Gel Dots

As the name suggests, PhysiFeet Invisible Gel Dots are clear gel dots, which can be placed over pressure areas where footwear rubs or irritates the foot. Easily placed, with sticky side to the shoe, the gel dots protect and cushion the foot. Thin and small, PhysiFeet Invisible Gel Dots are ideal for difficult and tight spots.




PRODUCT CODE AH-IHG

PhysiFeet Invisible Heel Guard

PhysiFeet Invisible Heel Guard discreetly sits on the counter of the heel to cushion and protect from footwear pressure and friction. The Heel Guard prevents slipping and rubbing as the contoured shape holds the heel comfortably in place. Easily placed, with the sticky side to the heel counter, the Invisible Heel Guard offers a lasting comfortable wear.



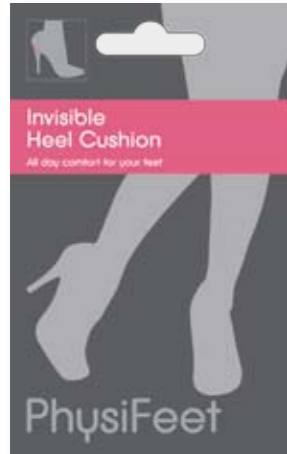

PRODUCT CODE AH-IHC

PhysiFeet Invisible Heel Cushion


The PhysiFeet Invisible Heel Cushion provides relief to tired and aching heels, especially for long hours of standing (and dancing) in fashion footwear. The thin clear gel cushion offers a non slip textured base, which fits easily into all shoes. Easily placed in the base of the heel area, the Heel Cushion is recommended for all high heel shoes.



PRODUCT CODE AH-GTP

PhysiFeet Gel Thong Protector

The PhysiFeet Gel Thong Protector is a perfect accessory for all straps found on thongs and sandals. The soft gel is designed and shaped to fit over the toe strap, to prevent friction and rubbing to the toes. The addition of PhysiFeet Gel Thong Protectors will alleviate rubbing, blistering and add comfort to the wearing of thongs and sandals.

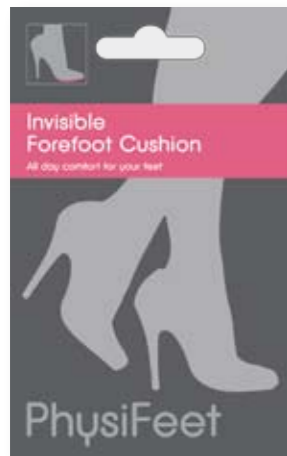

PRODUCT CODE AH-IFC

PhysiFeet Invisible Forefoot Cushion

The PhysiFeet Invisible Forefoot Cushion is for the tired and burning forefoot. When the pressure from high heels on the ball of the foot becomes too much, the thin cushion fits easily into place to offer relief. A non slip textured base prevents friction and shearing and fits easily sticky side down to cushion the forefoot.





DISPLAYS & CONSUMABLES

DISPLAYS

As part of the PhysiPod offer, we have developed a range of custom-built display stands designed to add value to your business. Professional, neat and attention grabbing, when housed in the display stand our colour coded pack designs present as a unified and intriguing point of interest for your reception area and clientele. Individual products are easy to distinguish from category to category and the fully adjustable hook and hole system allows you to configure the display to suit your practice.

Environmentally friendly, durable, strong and easy to assemble, our display stands are constructed of the highest quality cardboard and come in three styles to suit every working environment:

- Dual Sided Floor Stand
W49cm x H180cm x D43cm
- Wall Mount Stand
W65cm x H75cm X D20cm
- Spinning Table Top Stand
W30cm x H50cm x D30cm

Our competitive price structure enables you to make the PhysiPod range 100% profitable – at a minimum – and completely adjustable according to your competitor environment and retail requirements.

The PhysiPod Display Stand can either be purchased directly or comes to you free when you order a certain amount of product (call for further information).



CONSUMABLES

Allied Health Medical Supplies has sourced high quality clinical disposables and consumable products for your daily needs.

We have selected a large medical company with a proven history of the manufacturing and supply of clinical dressings and tapes. From this extensive range, we have selected the essentials, for your use, at a competitive price.



PhysiFix

Hypo-allergenic fixation tape. Light fabric allows for taping and securing dressings to sensitive skin.

- 5cm x 10m roll



PRODUCT CODE AH-PFR

PhysiFilm

A transparent non sterile adhesive film.

- 5cm x 10m roll
- water resistant film, ideal for covering all wound dressings requiring a water resistant barrier



PRODUCT CODE AH-PFIR

PhysiFix Dressing Roll

Adhesive roll with absorbent pad.

- hypoallergenic fabric wound dressing with low adherent pad
- Non sterile 6cm x 5m roll



PRODUCT CODE AH-PFDR

PhysiPlast

High elastic fabric adhesive dressing roll with low adherent pad.

- 6cm x 5m roll



PRODUCT CODE AH-PPI

PhysiPouch

Self sealing paper pouch with plastic film.

- Sizes: Small 90mm x 230mm & large 135mm x 260mm
- 200 per box



PRODUCT CODE AH-PP-S
PRODUCT CODE AH-PP-L

PhysiFix Wound Pads

Hypo-allergenic dressing with a low adherent wound pad.

- Ideal for dressing small areas
- 6cm x 7cm wound pad in single sterile pack
- 100 per box



PRODUCT CODE AH-PFWP

PhysiFilm Wound Pads

Transparent waterproof film with absorbent pad.

- Low allergy adhesive film & pad ideal for sensitive skin
- 5cm x 7cm wound pad in single sterile pack
- 100 per box



PRODUCT CODE AH-PFiWP

PhysiPod Face Sheets


PhysiPod offers a choice of two disposable face sheets to complement your medical practice and maintain the highest levels of hygiene for your clients.

A combination of innovative design and softer materials means increased patient comfort. Your equipment will also be better protected by the unique shape.

PhysiPod Face Sheets come in a unique and practical horse-shoe shape, allowing the edges to fold down into the table face hole, protecting your client's face from cross-contamination and your linen from soiling.

This design is excellent for clients who are somewhat claustrophobic when encountering the smaller circular face hole style cut-outs and x-cut style face sheets. It also allows your clients more space for their face and easier breathing from the prone position.

PhysiPod Face Sheets are a 'win win' for you and your clients!

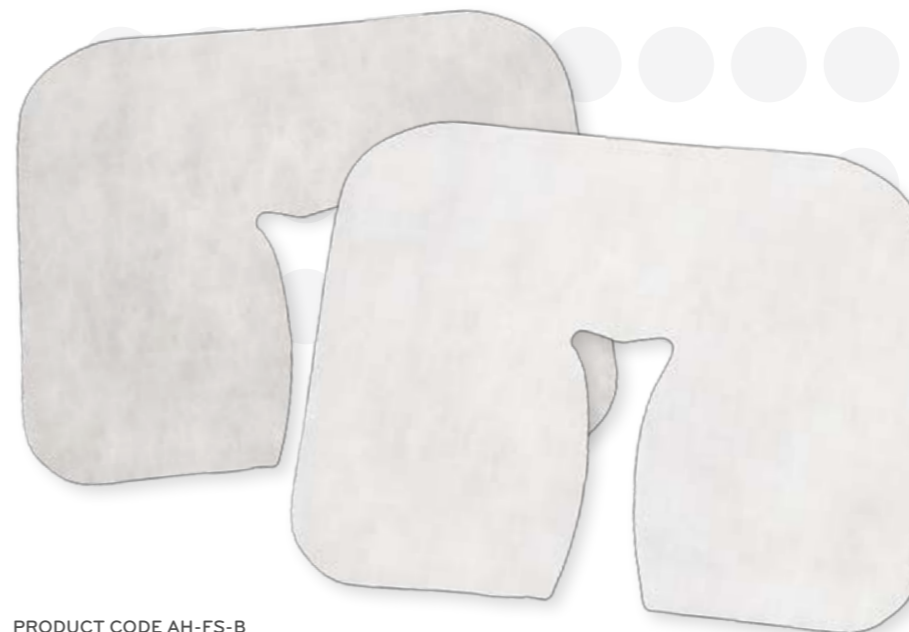

PRODUCT CODE AH-FS-B
PRODUCT CODE AH-FS-P

Budget

Our Budget face sheets are thin, but soft, pliable and very comfortable. They are strong and water resistant.

This will protect your bed and linen from the soiling effects of perspiration, make-up and also avoid any cross-contamination. The softness of this face sheet ensures your client is not left with the 'dimpled' imprint from traditional paper face sheets.

The sheets are sold in packs of 200. Size W34.5cm x H28cm.

Premium

PhysiPod Premium Face Sheets are the same size as the Budget sheets, but are made from a higher quality, 'spun lace' fabric, for an even softer, more plush sensation. This premium face sheet is a pleasure to lay on and, as with the budget version, will not leave the 'dimpled' imprint.

PhysiPod premium face sheets are pliable, water resistant and strong. They are sold in packs of 100. Size W34.5cm x H28cm.

INFORMATION & SUPPORT

Orders

For an order form please phone PhysiPod on +61 3 9366 6611, or email sales@physipod.com.au

Shipping

We use Australia Post and Couriers for interstate and oversease orders, with our goal being to send your goods to you in the quickest way possible.

Orders over \$250 (ex GST) are freight free within the Melbourne Metropolitan area.

Delivery Times

We will process your orders within 24 hours on a weekday (Mon-Fri) and endeavour to get your order to you as soon as possible. If items are currently out of stock, we will advise you via phone or email (please advise us your preferred means of communication).

Return Policy

To return or exchange an item, the following conditions apply:

- Items must not be used
- Packaging must be returned in a condition suitable for reselling
- A small shipping cost may be incurred when returning non-faulty exchange items based on the circumstances
- A 30 day limit exists; no return or exchanges will be accepted after 30 days from date of issue
- Deliver to: 266 Main Road East, St Albans, VIC 3021
- You must quote the invoice/Receipt No. (and provide evidence of this)

Faulty Goods

All goods returned as faulty will be inspected for a manufacturing defect. A full refund of the goods and the cost of shipping will apply if they are defective. For further clarification on this, please phone +61 3 9366 6611.

Please send all returns to:

ATTENTION:
RETURNS @ PHYSIPOD
266 Main Road East
St Albans, VIC 3021
Australia

Privacy and Security

Your contact details are required to ensure fast and accurate delivery of your order. These details are not passed on to any other party.

Your credit card payments are completely safe. We use the highest level of data encryption to ensure safe and secure transactions.

Contact Us

WEBSITE:
www.physipod.com.au

EMAIL:
enquiries@physipod.com.au
sales@physipod.com.au
vicki@physipod.com.au
nick@physipod.com.au


OFFICE ADDRESS:
266 Main Road East
St Albans, VIC 3021
Australia

WAREHOUSE ADDRESS:
152A Gaffney St
North Coburg, VIC 3058
(no sales conducted from this site)

PHONE:
+61 3 9366 6611

FAX:
+61 3 9364 2818



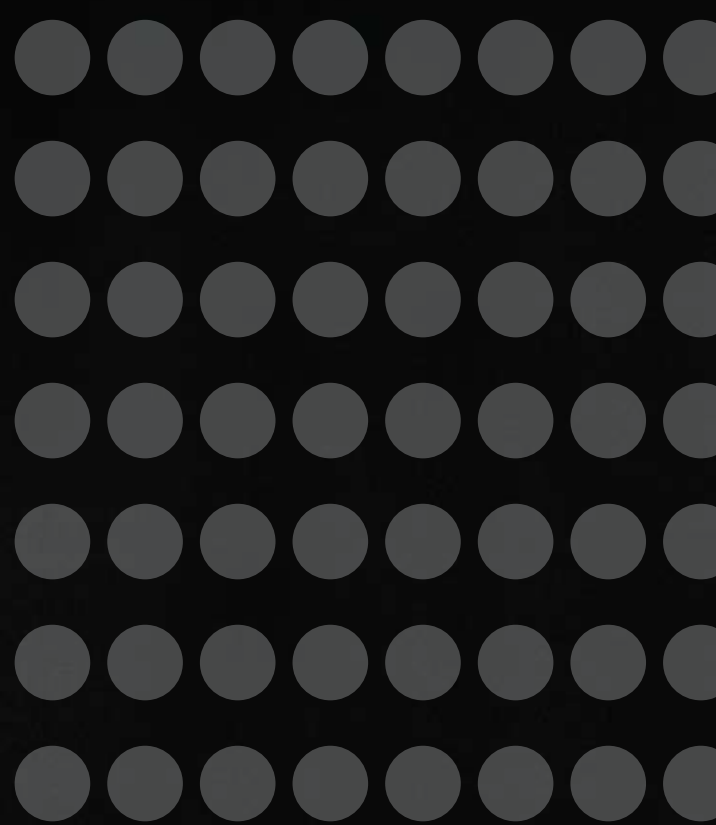

PhysiPod

BY PROFESSIONALS FOR PROFESSIONALS